

Cleaning Procedure for Pocketalker Systems

We recommend the following procedure for cleaning Pocketalker Amplifiers

1. Use an alcohol wipe or a solution of 3% hydrogen peroxide solution to clean the outside of the Pocketalker case and the earphone cord.
2. Can be carefully cleaned with "Hospital Cleaner Disinfectant Towels with Bleach, or alcohol" however, we cannot guarantee that the coloring / markings on the unit will not be affected as a result of multiple bleach applications. In addition it is crucial that "No Liquid" of any kind gets in the electronics of the unit, (i.e. Ear and microphone jack, volume knob, or tone control).
3. Replace the earphone foam pads with each new user, or use disposable sanitary headphone covers for each new user.
4. Do not spray or put liquid directly on the unit. Apply cleaning solution to a wipe or swab to clean item.

Caution: Avoid getting any liquid inside of the Pocketalker unit as it can damage the circuit board and will void the warranty.

800.843.3544 

952.943.2174 

info@williamssound.com 

williamssound.com 

10300 Valley View Rd. Eden Prairie, MN 55344