

# Sonic Alert

## Sonic Boom Alarm Clock & Bed Shaker

### Model SBB500SS



## OPERATION INSTRUCTIONS

**IMPORTANT** – Please read these instructions carefully before use and retain for future reference. See operating instructions on back.

- Use only the supplied adaptor to connect the unit.

**Warning:** To prevent fire or shock hazard do not expose this appliance to rain or moisture.

Before connecting to mains make sure that the requirements specified on the apparatus corresponds to the available power supply i.e. AC120V 60Hz or AC230V 50Hz.

- The ventilation should not be impeded by covering the ventilation openings with items such as, newspaper, table cloth, curtains, etc;
- No naked flame sources, such as lighted candles should be placed on the apparatus.
- It shall not be exposed to dripping or splashing and that no objects filled with liquids, such as vases, shall be placed on the apparatus.
- Do not place the product in closed bookcases or racks without proper ventilation.
- Where the mains plug is used as the disconnect device, the disconnect device shall remain readily operable.

## GENERAL DATA


Power Consumption: Adaptor AC9V~ 900MA)

UL listed transformer (Underwriters Laboratories) for USA

CE Approved transformer for Europe

**Made in China**

**[www.SonicAlert.com](http://www.SonicAlert.com) 1-800-566-3210 V/TTY**

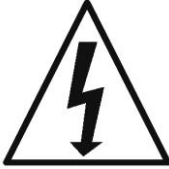

The Lightning flash and arrowhead within the triangle is a warning sign alerting you of dangerous voltage inside the product.

**CAUTION:**

TO REDUCE THE RISK OF ELECTRICAL SHOCK, DO NOT REMOVE THE COVER (OR BACK) FROM THE PRODUCT. NO USER SERVICEABLE PARTS INSIDE. REFER SERVICING TO YOUR SONIC ALERT DEALER.



The exclamation mark within the triangle is a warning sign alerting you of important instructions accompanying the product.

**WARNING: TO PREVENT FIRE OR SHOCK HAZARD, DO NOT EXPOSE THIS APPLIANCE TO RAIN OR MOISTURE.**

- Attention should be drawn to the environmental aspects of battery disposal.
- this indicating separate collection for electrical and electronic equipment.

***SONIC ALERT'S 1 YEAR LIMITED WARRANTY***

The Sonic Boom Alarm Clock Model SBB500ss is warranted Against manufacturing defects in materials and workmanship For (1) year from the date of purchase. Within this period Sonic Alert will repair or replace at our option the SBB500ss without Charge for parts and labor. Simply send the SBB500ss (postpaid) And a copy of your sales slip as proof of purchase to:

**Sonic Alert Inc., 1050 E. Maple Road, Troy MI 48083, USA.**

## **Sonic Alert**

**Made in China**

**Sonic Alert 1050 East Maple Rd. Troy, MI 48083**

**[www.SonicAlert.com](http://www.SonicAlert.com) 1-800-566-3210 V/TTY**

INSTRUCTIONS  
 ALARM CLOCK WITH BED SHAKER  
 MODEL SBB500SS

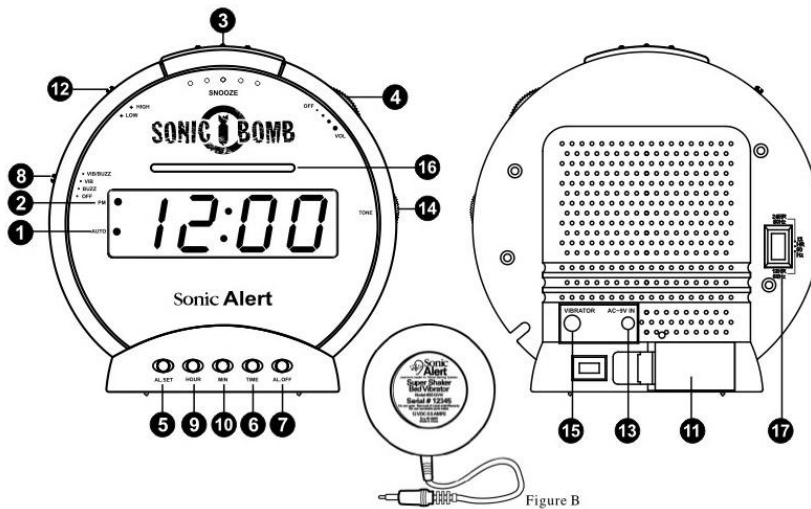


Figure B

Features show on diagram above:

- |                    |                          |                               |
|--------------------|--------------------------|-------------------------------|
| 1) Alarm Indicator | 7) Alarm Off Button      | 13) AC Adaptor                |
| 2) PM Indicator    | 8) Alarm Function Switch | 14) Tone Control Wheel        |
| 3) Snooze button   | 9) Hour Button           | 15) Vibrator Output           |
| 4) Volume Wheel    | 10) Minute Button        | 16) Flashing Light Display    |
| 5) Alarm Set       | 11) Battery Compartment  | 17) Hour Format & Power Cycle |
| 6) Time Set Button | 12) Dimmer Switch        |                               |

**Setting the correct time –**

- 1) Press and hold the Time Button (6) at the same time press the Hour Button (9) to advance the clock display rapidly until the hour is correct.
  - 2) Press and hold the Time Button (6) at the same time press the Minute Button (10) to advance the clock display rapidly until the hour is correct.
- A lighted dot (20 in the upper left of the clock display indicated the time is PM. For AM time indication, there is not lighted dot.

**Setting the alarm time –**

- 1) Simply follow the instructions mentioned under “Setting the correct time“ except that you will have to press the Alarm Button (5) instead of the TIME Button.
- 2) Make sure the alarm time is correctly set for AM or PM, the lighted dot (2) indicated PM. Be sure to release the HOUR and MIN set buttons before releasing the ALARM button.

**Waking to Alarm –**

- 1) Set Function Switch (8) to choose „VIB“, „BUZZ“ or „VIB / BUZZ“.
- 2) Set desired volume (4) for buzzsound.
- 3) Set alarm wake time per instructions for setting alarm time above.

### **Battery Backup –**

To use the battery backup, you can install a 9-volt alkaline battery in the back of the clock located under the battery cover (11). Battery not included. Note the LED display will not show during power failure but the clock will still be running and keeping proper time until the power comes back.

**NOTE:** For the best results it is recommended that the battery be changed every 6-8 months. If the clock is not used for a while, it is recommended to remove the battery to avoid leakage.

### **Shutting Off Alarm –**

Alarm can be shut off temporarily for the day or shut off permanently.

- 1) When the alarm goes off, simply press the AL off button (7) the alarm will shut itself off for the day and come back on the next day at the same time if the alarm setting has not changed.
- 2) To shut off the alarm permanently, slide function switch (8) to off position. Alarm will not come on until Function Switch (8) is set to „VIB“ or „BUZZ“ or „VIB/BUZZ“ position.

### **Snooze Operation–**

SNOOZE button (3) – press after alarm has sounded. The alarm will be turned off for 9 minutes for extra sleep. The alarm will come back again after 9 minutes. You may repeat many times up to 1 hour.

### **Alarm –**

Set the alarm time you wish and slide the VIB/BUZZ/OFF (8) switch to the desired position. Set to VIB for bed shaker and BUZZ for sound, or VIB/BUZZ for both.

### **Tone –**

Adjust the tone by rotating the tone wheel (14) located on the right hand side just below the volume wheel (4) until you reach desired level.

### **Volume –**

To adjust the volume rotate the volume wheel (4) located on the upper right hand side.

### **Bed Shaker / Vibrator – Figure B**

- 1) Attach the bed shaker jack to the back of the clock at VIBRATOR input (15).
- 2) To turn vibrator slide function switch (8) to VIB or VIB / BUZZ position.
- 3) Place vibrator under mattress or pillow.

### **AC 9V Power Input –**

Attach the AC power supply input to the side of clock at AC9V IN (13) next to the VIBRATOR input (15), smaller input of the two. Important: do not confuse AC input with vibrator input.

### **Hour Format and Power Cycle (17) –**

Please select the appropriate settings for use. For USA please select 12/60Hz; for Europe please select 24/50Hz; for UK please select 12/50Hz. Please note the incorrect selection of power cycle will result in incorrect time.

**Made in China**  
**Sonic Alert 1050 East Maple Rd. Troy, MI 48083**  
**[www.SonicAlert.com](http://www.SonicAlert.com) 1-800-566-3210 V/TTY**